# Europa B EVO - B EVO24

B EVO • ART. 2033S2  
B EVO24 • ART. STGA099

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EN

Europa B Evo is an autoclave that has been developed jointly by the Tecno-Gaz research office and important external development facilities. Europa B Evo is indispensable for all medical and dental facilities and in all fields where safe sterilization of all materials is required: hollow, porous, solid, wrapped and unwrapped. Europa B Evo continues the success that Tecno-Gaz has achieved over the years, in fact, thousands of professionals use Tecno-Gaz autoclaves and are very satisfied.

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### P.I.D. System

P.I.D. means Proportional Integral Derivate and represents the apex of closed loop controls. The structure of this technologically advanced control is simple and flexible. P.I.D. ensures a constant control and monitoring of the whole operating system of the autoclave, acting automatically in case of anomalies or incorrect values in technical and electronic parameters. **This device behaves like a "real technician" with real-time diagnostic and intervention capability. This solution reduces anomalies and warnings, granting maximum reliability and minimising the machine inactivity times.**

### Safety on the door

We have applied three safety devices on the autoclave door:

- **The first device is a mechanic and electromagnetic one:** it triggers when the sterilization cycle begins and completely blocks the door opening. This device can be removed at the end of the cycle by pushing the device-unlocking button, but only if positive pressure is near zero.
- **The second device is an electrical one:** it triggers automatically when an attempt to force the door open is detected.
- **The third device is dynamic** and acts on the structure of the gasket.

### Safety devices in place

- Certified safety valve.
- Ped on sterilization chambers.
- Insulation of the chamber with certified ceramic material.

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### Barometric alignment

**Automatic barometric alignment** is provided from the first time the device is switched on, without any technical intervention.

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### Overnight cycles

**Automatic overnight cycles** with anti-condensing system and automatic switch-off, with memory effect.

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### Provision for mains water supply

**The osmosis system mod. Pura is an extremely compact** piece of equipment for osmosis, created to be directly connected to the autoclave, guaranteeing a constant supply.

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### Built-in printer

**Built-in printer for the registration of cycles carried out.** The printer includes:

- date, time
- cycle number
- serial number of the device
- minute by minute cycle data
- outcome of the cycle